WELLSYSTEM MEDICAL

WELLSYSTEM RELAX

Montageanweisung · Assembly Instructions

wellsystem™

wellsystem™

801285 / Index „a“ / de / en / 01.2004

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Table of Contents

How to use the installation instructions 17

Meaning of symbols 18

Safety 20

Export 20

Transport and transport damage 21

Maintenance and care 21

Deenergise device and secure against being switched on again 22

Dismantling and assembly instructions 24

Dismantling

Transport 27

Transport units 28

Dismantling the panels and frame 29

Dismantling the pump 30

Dismantling the tank 31

Dismantling the frame 32

Assembly

Assembling the frame 33

Installing the tank 34

Installing the pump and connecting it up 35

Aligning the massage unit: Empty tank 36

Filling the tank 37

Aligning the massage unit: Filled tank 38

Connecting up the cooling system 38

Fitting the control panel 39

Fitting the panels 40

Technical Data 41

Appendix: Warning sticker on the equipment 43

Index 47

Note:

Technical changes are reserved with respect to the illustrations and specifications in this instruction manual. Reprinting and reproduction, in whole or in part, of this manual is only permitted with our previous written approval and with specification of the source.

Please read and observe ...

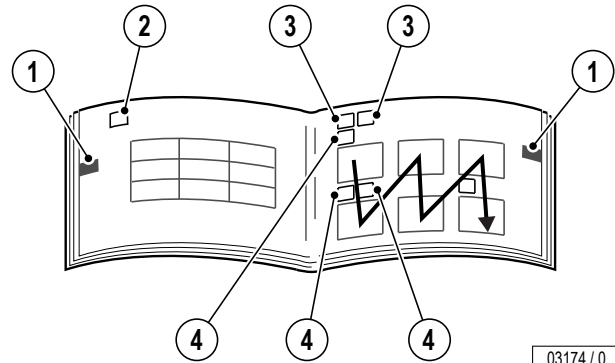
How to use the installation instructions

The chapters "Dismantling", "Installation" and "Technical Data" in this instruction manual primarily contain graphics and symbols, but only very little text. Important general information and safety precautions are provided here in the first chapter.

There are several types of symbols:

- Chapter symbols (1) (e. g. Dismantling, Installation). These symbols have a black background and are positioned on the outer edge of the page. This chapter symbol can also be used as the page heading (2).
- Symbols in the top line (3) apply to the entire page. These may be headings or important information.
- Symbols directly over a graphic (4) refer to the work step shown in the graphic.

The meaning of the symbols is explained beginning on Page 18 in this chapter.

Meaning of symbols

Danger notes:

Danger!

This safety note – warning triangle with the word “Danger” – points out that danger for persons can be expected here (danger to life and limb).

Example:

Extreme danger!

Warning! High voltage!

Caution!

This safety note – warning triangle with the word “Caution” – points out that danger for devices, materials and the environment can be expected here.

Important information:

Note:

This symbol does not identify safety notes, but states information for a better understanding of the processes.


General symbols

Dismantling	
Assembly	

Danger!

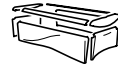Deenergise device – disconnect from power supply

Panels

Tank

Pump

Please read and observe ...

Wear industrial safety gloves

Description

Visual inspection

Two persons are required for this work step.

Unscrew/release screw

Tighten/lock screw

Unscrew/release screw

Tighten/lock screw

Unscrew/release screw

Tighten/lock screw

Technical Data

Safety

Danger!

Assembly and electrical connection must comply with the national regulations¹⁾.

Assembly, installation, upgrading or repair of the device may only be carried out by specially qualified personnel who have been trained by JK-Global Service.

- No safety equipment (e.g. switches) or safety notes may be removed or deactivated which can impair the safe operation of the device!
- You must abide by all hazard signs and safety instruction attached to the unit during dismantling and assembly (see also Page 43).
- The person responsible for assembling/installing the unit is obliged to release the unit for use in perfect condition only.
- **The massage unit must never be run when it is empty!**
- A water cooling system must be connected to the unit if the utilisation period will exceed 3 hours/day. The cooling water drainage hose must never be removed.
- We recommend using a water shut-off valve (water flow control).
- The unit must be made voltage-free before any dismantling or assembly work is carried out on the electrical units (see Page 22). Exceptions are permissible only for function tests!

In Germany: VDE regulations

- Dismantling as well as assembly is only permitted in the sequence stipulated in this manual!
- Any thermal therapy units (microwave therapy units, etc.) located in the vicinity might cause electro-magnetic interference. Please let customer service know if this is the case (see Page 2).

Export

We emphasise that these devices are intended for the European market and must not be exported to nor operated in the USA or Canada! The manufacturer shall assume no liability whatsoever in case of non-observance! We explicitly emphasise that high liability risks may occur for the exporter an/or the operator in case of non-observance.

Please read and observe ...

Transport and transport damage

Attention!

If the massage unit has to be transported with the pump fitted, the transport safety device must be fitted to the pump – see Page 27.

Please inform the owner if you discover any transport damage when unpacking and installing the unit.

Please take note of the following information, so that any damage can be taken care of quickly and without any complications arising:

- Any recognisable damage must be reported in writing within a period of 4 working days – this includes the delivery date and being received by the transport company – to the transport company that delivered the shipment.
- The resulting damage must be made good by the transport company that delivered the shipment as, according to the general terms and conditions of the transport company, the transport risk is assumed by the person who ordered the equipment. Naturally, we will fully support you in asserting your justifiable claim.

Maintenance and care

Extreme danger!

The device must be deenergised when performing maintenance work requiring the opening of the device.
See Page 22.

Remount any protective devices (e.g. filter panels) after completing work.

Caution!

Only use genuine spare parts of the same type!
The use of other parts voids the CE conformity!
Any liability is excluded in case of damage or injury proven to be the result of the use of non-genuine spare parts.

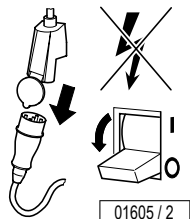
Deenergise device and secure against being switched on again


Extreme danger!

If work is to be performed on the devices, they must be deenergised. That means that all live lines must be switched off.

Switching off the device alone is insufficient, as certain points may remain energised. Therefore, switch off fuses before performing any work and – if possible – remove them.

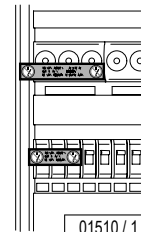- In case of circuit breakers which cannot be screwed out, a strip of adhesive tape may be applied over the actuation lever with the wording "Do not switch, danger!"

Extreme danger!

Severe accidents can occur if the device is mistakenly switched on again.

Immediately after deenergising, secure all switches or fuses used for deenergising against being switched on again.

- Lock the fuse box with a padlock.

A prohibition sign with the text:

"Work in progress!"

- "Place:"
 - "Sign may only be removed by:"
- must always be securely mounted immediately.

01511 / 1

Extreme danger!

Prohibition signs must not be hung on or touch live parts.

Please read and observe ...

Environmental protection information

Environmental Declaration – The JK Corporate Group

The JK corporate group is subject to the strict regulations of EC Directive 761/2001 and the standard DIN ISO 14001, and undergoes regular internal and external environment audits performed by trained auditors.

Used water disposal

The massage unit is filled with water from the normal supply line. The water can be disposed of in the public drainage system after it has been used.

Packaging

All packaging consists of 100 % recyclable materials. Packaging brought into circulation by the JK Corporate Group that is no longer required can be returned to the JK Corporate Group. Your agency partner or dealer will be happy to advise you.

Disposal of recyclable materials

The device has been produced of recyclable materials. When being scrapped later, the device must be disposed of properly. The JK Corporate Group will provide you with information on the content or potential hazards of the materials used.


Dismantling and assembly instructions

Note:


The illustrations and text apply to both units, even if only one is shown unless explicit differentiation is made between **wellsystem™_Medical** and **wellsystem™_Relax**.

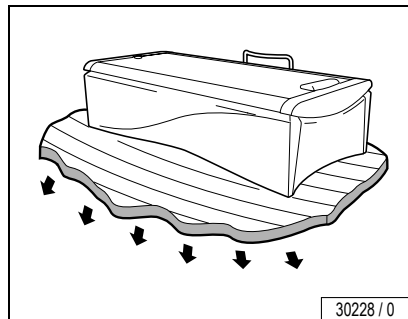Installation site

When installing the unit, you must ensure that the floor and ceiling in the commercially used premises are suitable for taking a minimum load of 1,500 N/m² when installing the unit.

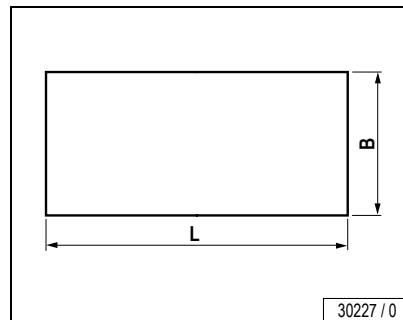If the actual maximum load exceeds that value, then a special certificate must be obtained before you can use these premises.

The massage unit can be placed on multiplex board or chipboard¹⁾ in order to better distribute the pressure from the adjustment feet over the floor.

L = 2,150 mm

W = 1,000 mm

1) optional

Tools and outfitting

Danger!

Always wear industrial safety gloves, when working on massage equipment that is already operating, as it is possible to make contact with the water being used (e.g. when changing the nozzle trolley).

Always use an antiseptic cleaner to clean your hands afterwards, e. g. Skinman Soft SMS 50.

The following tools are required for the dismantling / assembly work:

- Cross-headed screwdriver 2
- Cross-headed screwdriver 3
- Double ended SW 8-19 mm spanner
- 5 and 6 mm Allen-keys
- Water pump pliers or SW 46 and 55 ring spanners
- Ratchet with SW 7, 8, 10 and 13 socket sets
- Size 4 blade-screwdriver
- Spirit level, at least 800 mm long
- 1/2" water hose with "Gardena" hose-piece
- Sliding agent (e. g. soap)

Note:

If a cooling system has to be attached (daily operation will be more than 3 hours), then you require 2 additional high pressure hoses (10 mm, 3/8").

Return: 10 bar

Supply: 60 bar

Electrical connections

- Only an authorised specialist company is permitted to make the on-site electrical connections (equipment connection socket)!
- You must abide by the regulations.
- We recommend installing a selective RCD (Residual Current Device; nominal fault current 30 mA).
- Only one massage unit should be connected to each RCD.
- **wellssystem™_Medical**: A line impedance of 0.14 ohms must be guaranteed for pulse massaging.
- The electrical installation must have a freely accessible, all-pole isolating device (main switch) fitted externally on the site side.
- The unit assignment can be seen from the inscription.
- **wellssystem™_Relax**: The connection is made via a plug with an earthing contact. The socket must always be accessible.
- **wellssystem™_Medical**: The equipment must be permanently connected to the public power supply via an all-pole main switch. The connections and the associated fuses must be readily accessible.
- The cross-section of the lines must be chosen in compliance with the national regulations¹⁾, e.g. these must be taken into consideration:
 - the length of the lines
 - the connection lines being used.
- A supply line is required for each unit.

1) In Germany: VDE 0100

- Data and/or control lines are to be laid with a minimum distance of at least 10 cm from the mains cables.
- The mains supply lines must be laid before the unit is installed.

Test run/Commissioning

- Has it been filled with sufficient water?
- If daily operation will be more than 3 hours: has a cooling system been attached?
- Is the heating connected up?
- Are you sure that there are no leaks coming from the tank or pump?

If all questions were answered with “Yes”, then you can start the test run or operate the unit.

- Switching the unit on – see operating manual.

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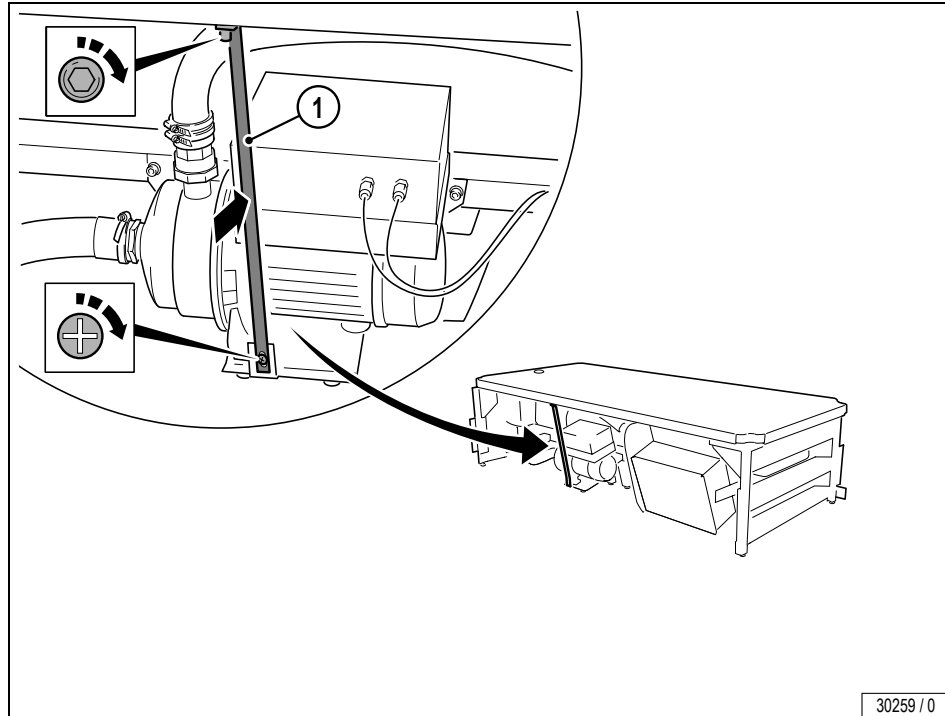
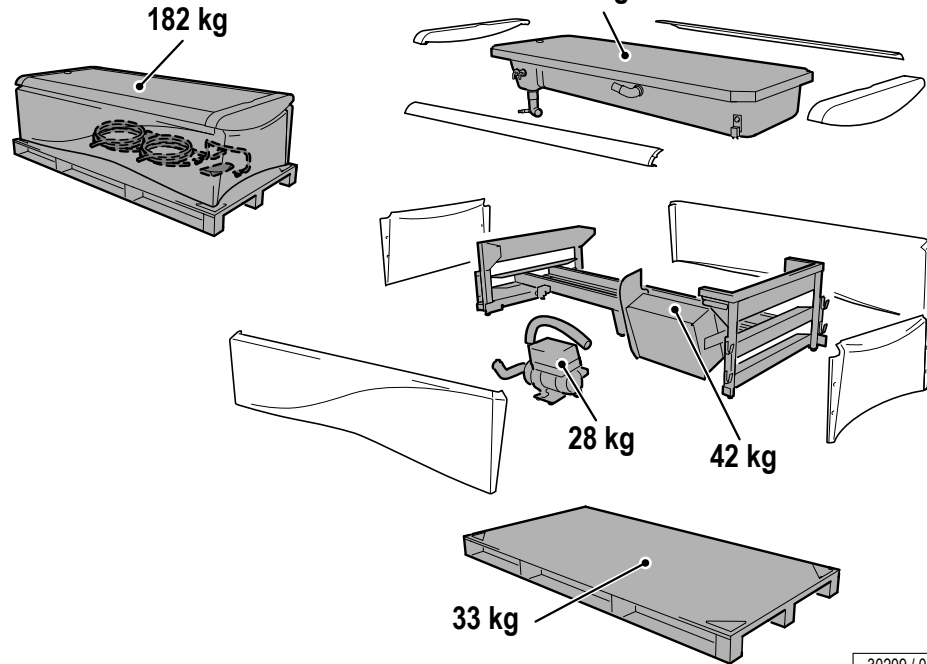
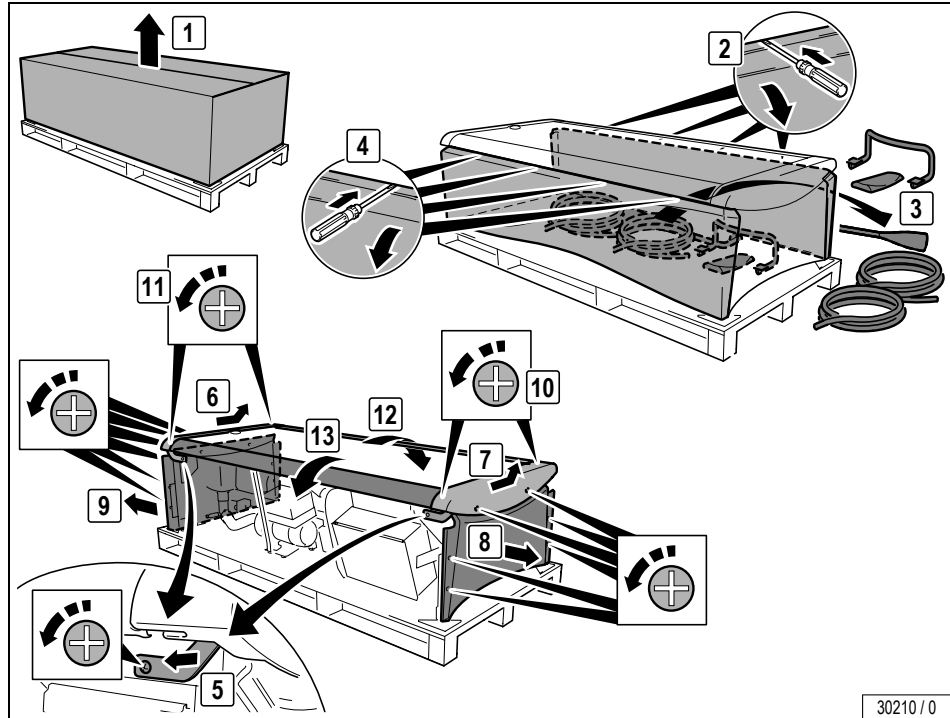
GB Dismantling the panels and frame


Demontage
Dismantling

30210 / 0

D Pumpe ausbauen

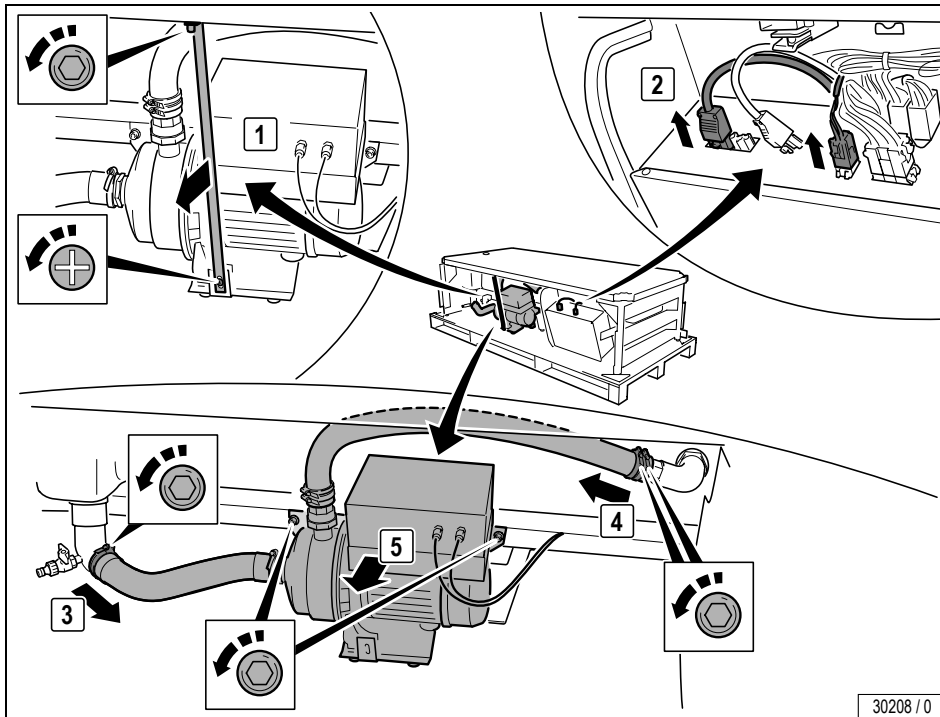GB Dismantling the pump


30208 / 0

Demontage
Dismantling

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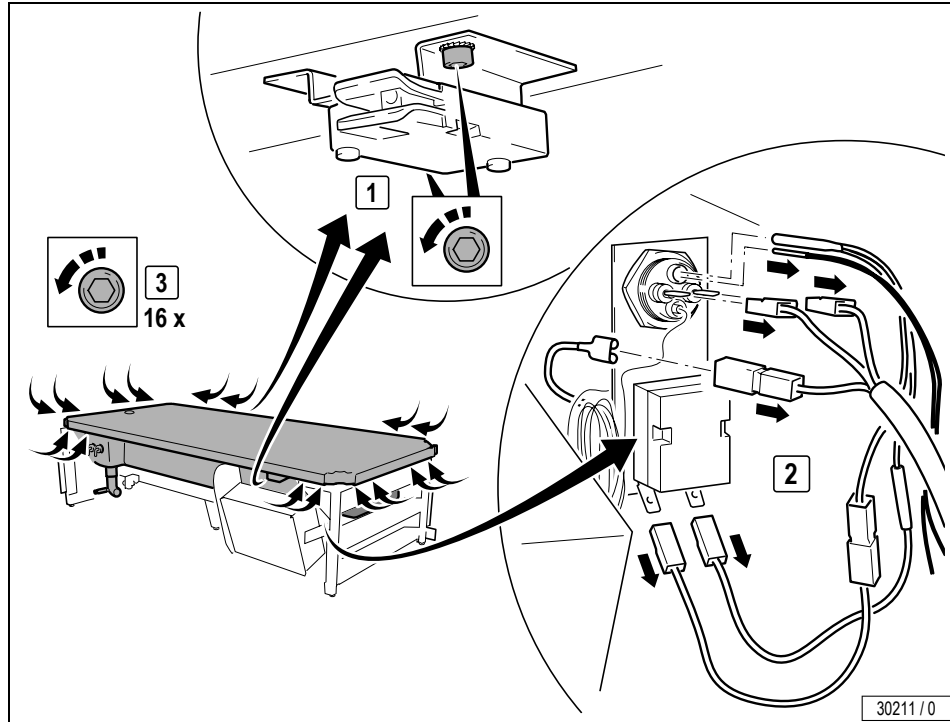GB Dismantling the tank


Demontage
Dismantling

D Gestell demontieren

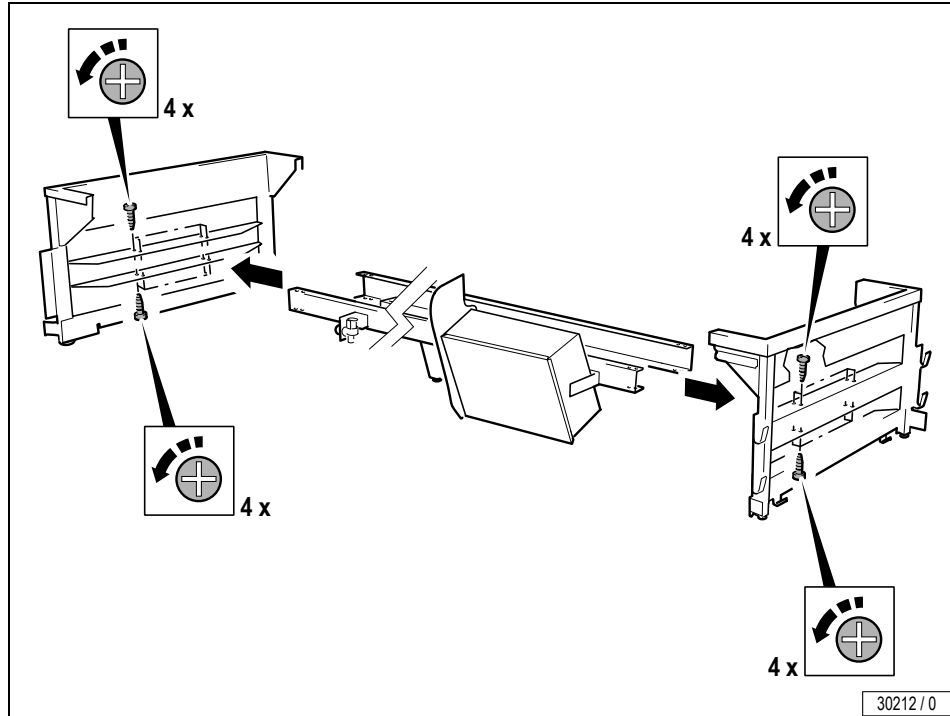GB Dismantling the frame


Demontage
Dismantling

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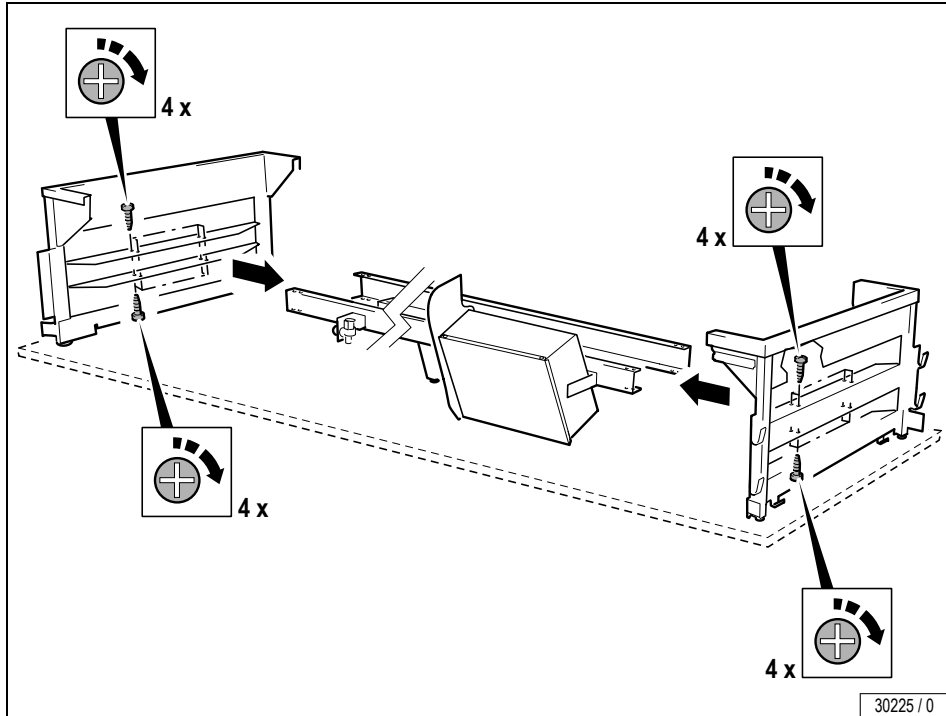GB Assembling the frame²⁾

1) optional: Bodenplatte

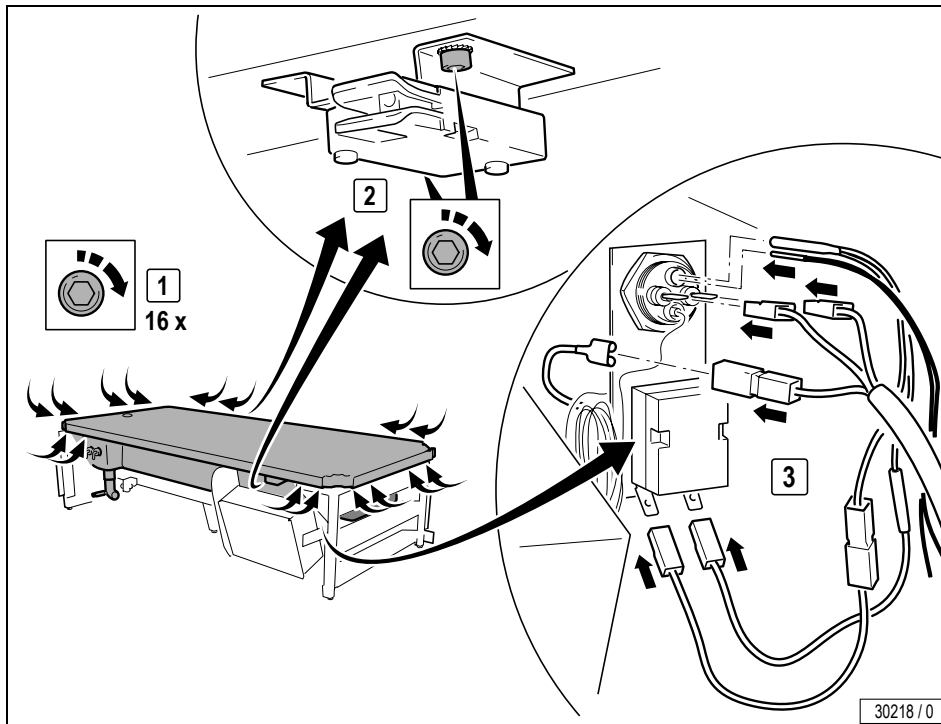
2) optional: floor plate

Montage
Assembly

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GB Installing the tank

30218 / 0

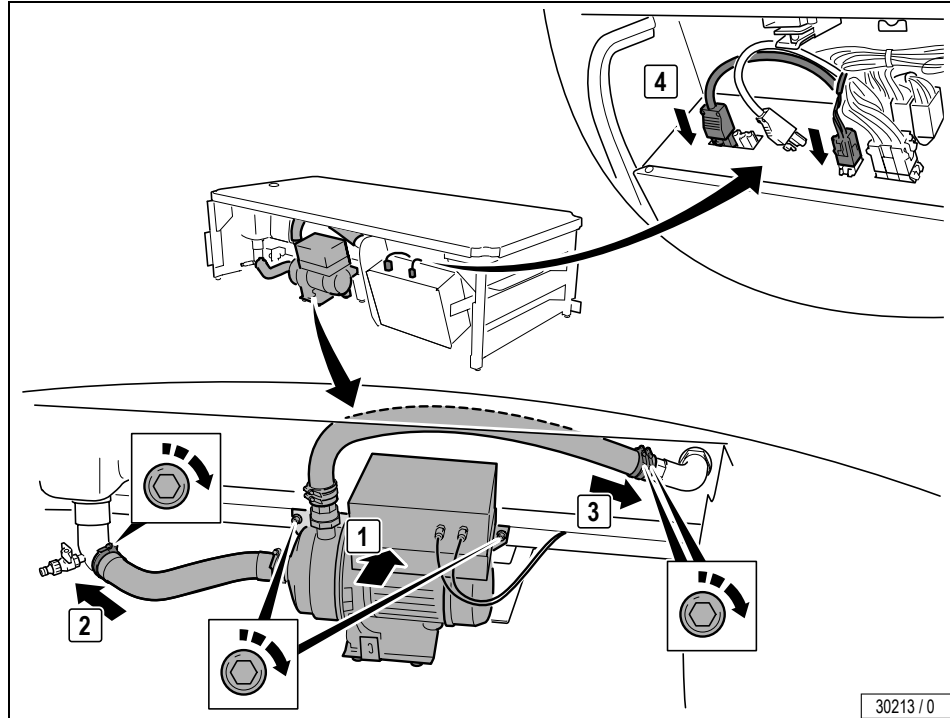
Montage
Assembly

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GB Installing the pump and connecting it up

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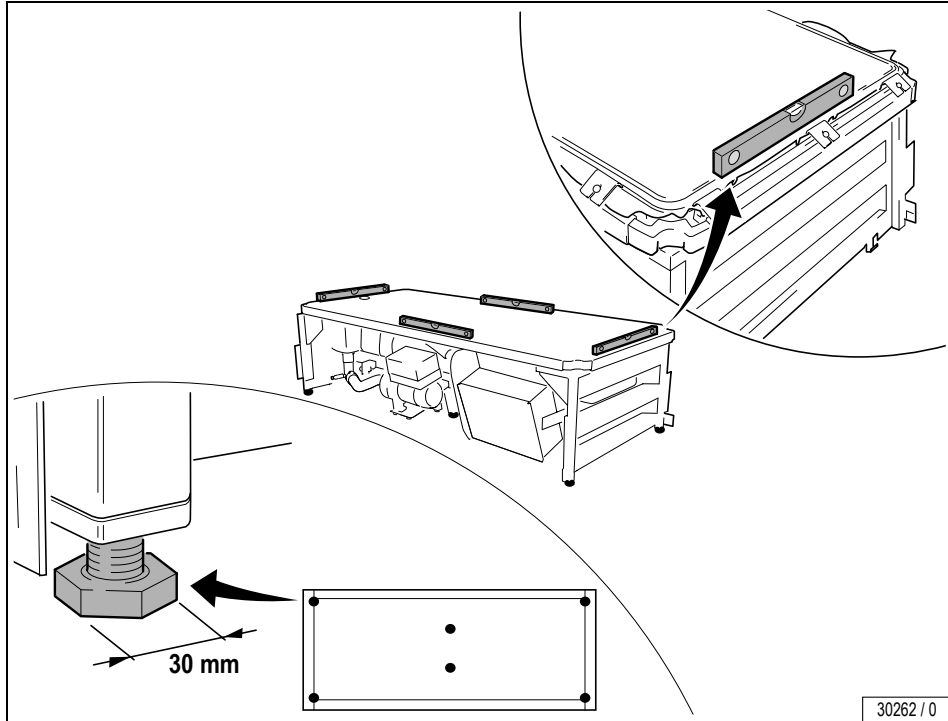
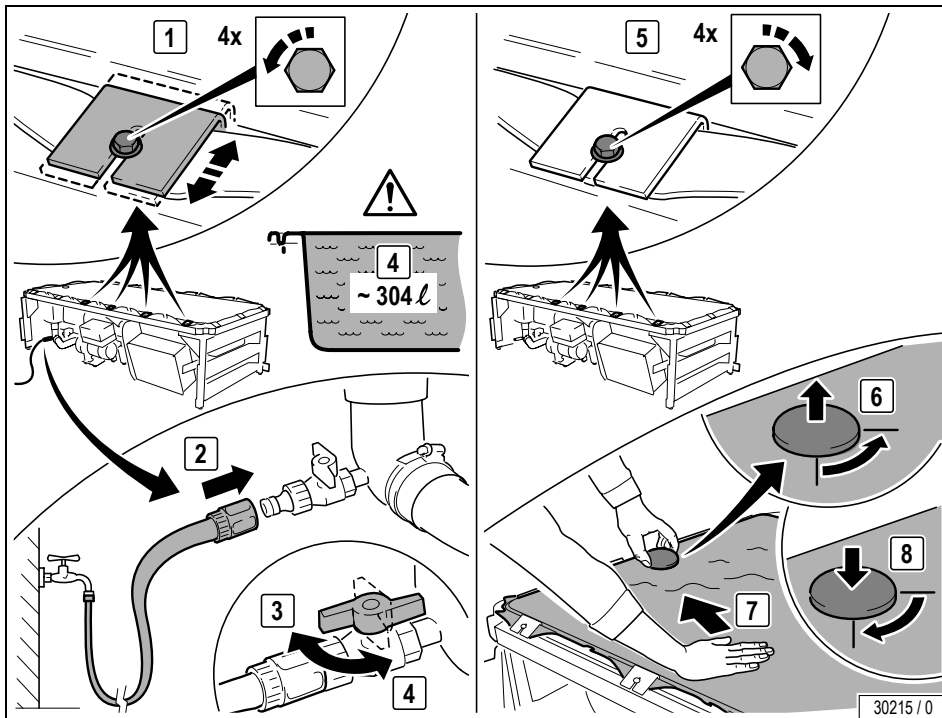
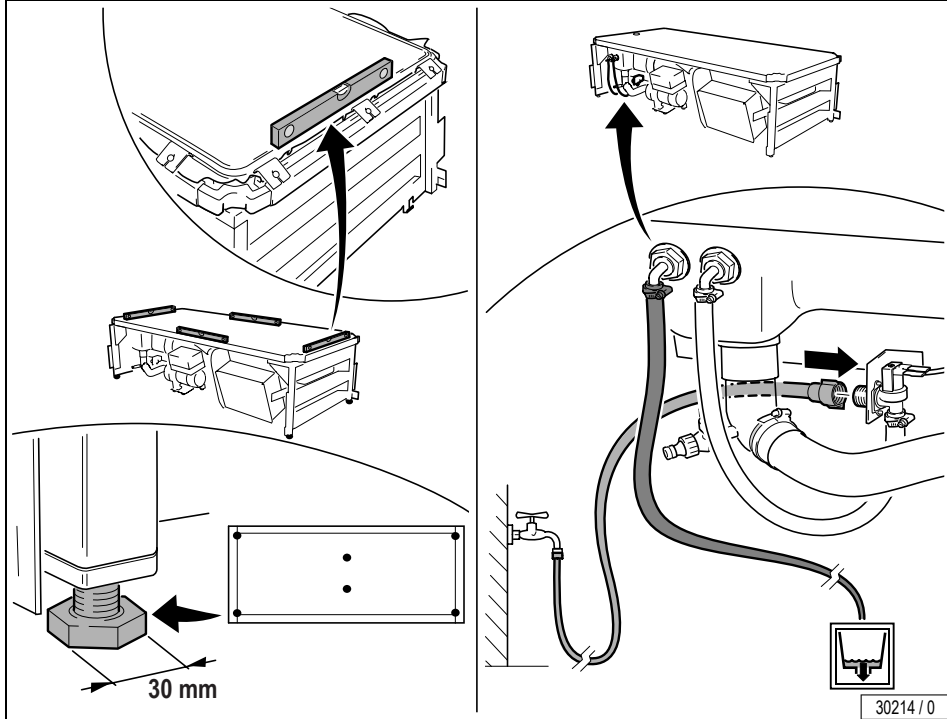
GB Aligning the massage
unit: Filled tank
Connecting up the cool-
ing system¹⁾

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Wellsystem GmbH